

# AED SOLUTIONS

FOR COMMUNITIES & CHURCHES



American AED, Inc.  
405 N.W. 10th Terrace - Hallandale Beach, FL 33009  
Toll Free: 1-800-927-9917  
Tel: 954.458.6618 | Fax: 954.241.5577

[www.AmericanAED.com](http://www.AmericanAED.com)  
Copyright © All Rights Reserved.

## Tragedy Can Strike When You Least Expect It, AEDs Can Provide Immediate Life-Saving Treatment

Sudden cardiac arrest can strike at any time—during work, at a sporting event, or while at church. In a typical community setting, victims of sudden cardiac arrest rarely survive because they do not have immediate access to life-saving treatment, but studies show that more than half would survive if an AED and CPR had been used within five minutes of cardiac arrest.

Because a large part of the American population attends regular religious services, there is a good chance that many people could experience sudden cardiac arrest while at church. By placing AEDs in churches, congregation members are able to act as first responders and prevent unnecessary deaths in their communities. Churches can prevent unnecessary tragedy during religious services, community gatherings and special events such as baptisms or weddings. By providing AEDs in churches, church members can begin the life-saving process while waiting for emergency responders to arrive and therefore increase the chances of survival for victims of sudden cardiac arrest.

### WHY YOU NEED AN AED TO PROTECT YOUR CONGREGATION

Here are some of the reasons why more and more churches are making AEDs a part of their emergency response plans:

- ▶ Many people who experience sudden cardiac arrest die as a result, mainly because of the delay in treatment due to waiting for emergency responders.
- ▶ Many churches provide spiritual assistance in today's world, now with AEDs widely available and financially accessible, churches can also provide life-saving assistance and congregation members can be assured of their spiritual and physical safety.

## About The AED Defibrillator Included In The Church Package:

### OnSite Automated External Defibrillator

Weighing just 3.3 lbs., this small and lightweight automated external defibrillator (AED) can be easily carried to the patient's side. Using clear and calm voice instructions, the OnSite guides you, the first caregiver on the scene, through each step of the rescue process. The OnSite uses Philips proven technology for heart rhythm assessment (SMART Analysis) and defibrillation energy delivery (SMART Biphasic), and can be used to treat adults, children and infants.



### Ready When Needed – Easy to Use

- ▶ The OnSite is powered by a disposable lithium battery and will perform daily, weekly, and monthly self-tests of its internal circuitry and shock delivery system. As long as the flashing green “Ready” light is blinking, you can be certain the Onsite is ready-to-go if needed.
- ▶ In the event of a sudden cardiac arrest, simply pull the lever and the OnSite will begin providing voice instructions to guide you. Once the electrode pads are placed on the patient the OnSite’s special sensors determine if a shock is needed, based on its automated assessment of the patient’s heart rhythm.
- ▶ If a shock is needed, the voice prompts direct you to press the flashing orange “Shock” button. Then the OnSite delivers a low-energy biphasic current.
- ▶ If a shock is not needed, the OnSite instructs you to perform cardiopulmonary resuscitation (CPR) if necessary. While performing CPR, the voice prompts coach you on the frequency and depth of your compressions.
- ▶ The OnSite also reminds you to call emergency medical services (EMS). When they arrive should they need a summary of care, it can be retrieved from the OnSite’s internal memory.



# PHILIPS

sense and simplicity

## American AED Church Package

Ships with everything you need to set-up your own AED station.

Includes a Wall Cabinet and Double Sided AED Sign to store, display, and quickly locate the AED when needed.

### Your Complete Church AED Package Includes:

- |  |  |
|--|--|
| 8 Year Warranty                                | 1 FREE AED Demonstration & Training DVD                  |
| 1 Pair Of Adult Smart Pads                     | 1 FREE Premium AED+CPR Responder Kit (\$50 value)        |
| 1 Battery With 4 Year Warranty                 | 1 FREE Inspection / Maintenance Tag                      |
| 1 User Manual                                  | 1 DOUBLE SIDED FLANGED AED SIGN                          |
| 1 Quick Use Guide                              | 1 AED STORAGE WALL CABINET                               |
| Philips HeartStart Review Express              | 1 AMERICAN AED INSTRUCTIONAL WALL AED POSTER             |
| 1 FREE Philips HeartStart OnSite Carrying Case | AMERICAN AED Lifetime Maintenance Notification & Support |
| MSRP: \$1,512.99                               | FREE UPS SHIPPING  |

Exclusive Offer:	\$1275.00
Church Discount:	-\$100.00
No Sales Tax:	\$0.00
Free UPS Shipping:	\$0.00
<b>Grand Total:</b>	<b>\$1175.00</b>

**#1 Best Selling AED**  
Package For Churches  
More Than 10,000 Units Sold!

Optional:  
Infant/Child Smart Pads, \$108.00

Note: If your organization has young children, 8 years old or younger and/or 55lbs and under, consider adding a set of infant / child smart pads. (Adult pads are used for adults or children 8 years old and older and/or 55lbs and OVER. Infant / child smart pads are for children 8 years old or younger and/or 55lbs and UNDER).

**ORDER FORM**

Use this form to order by check or money order.  
To order by credit card call 1-800-927-9917



**Your Complete Church AED Package Includes:**

- 1 Brand New Philips HeartStart OnSite AED Defibrillator
- 8 Year Warranty
- 1 Pair Of Adult Smart Pads
- 1 Battery With 4 Year Warranty
- 1 User Manual
- 1 Quick Use Guide
- Philips HeartStart Review Express
- 1 FREE Philips HeartStart OnSite Carrying Case
- 1 FREE AED Demonstration & Training DVD
- 1 FREE Premium AED+CPR Responder Kit (\$50 value)
- 1 FREE Inspection / Maintenance Tag
- 2 FREE AED "Equipped Facility" Decal / Sticker
- 1 DOUBLE SIDED FLANGED AED SIGN
- 1 AED STORAGE WALL CABINET
- 1 AMERICAN AED INSTRUCTIONAL WALL AED POSTER
- AMERICAN AED Lifetime Maintenance Notification & Support

Exclusive Offer:	\$1275.00
Church Discount:	-\$100.00
No Sales Tax:	\$0.00
Free UPS Shipping:	\$0.00
<b>Grand Total:</b>	<b>\$1175.00</b>

**FREE UPS SHIPPING**

SKU	Description	Unit Price	Qty.	Ext. Price
M5066A-CHURCH	American AED Church Package	<b>\$1175.00</b>		
Wall Cabinet Option	Upgrade Wall Cabinet With Audible Alarm & Alert Light	\$20.00		
Wall Cabinet Option	Upgrade Wall Cabinet With Audible Alarm & Strobe Light	\$90.00		
Optional Child Pads	Philips HeartStart OnSite Infant/Child Electrode Pads	\$108.00		
To order by check or money order:		<b>Subtotal:</b>		
Mail this form along with a check made payable to American AED, Inc.		FL 6% Sales Tax:		<b>N/A \$0.00</b>
Mail To: 405 N.W. 10th Terrace - Hallandale Beach, FL 33009-3105		Shipping:		<b>FREE \$0.00</b>
Your order will be processed and shipped within 24hrs upon receiving your order. Please allow 2 to 4 business days for UPS delivery		<b>Grand Total:</b>		

**Customer Shipping Information:** (Please No P.O. Box as we ship via UPS)

Company / Organization: \_\_\_\_\_

Attention: \_\_\_\_\_

Address: \_\_\_\_\_

City, State, Zip Code: \_\_\_\_\_

Telephone: \_\_\_\_\_

Email: \_\_\_\_\_  
(Please provide an email address so we can email you the receipt and UPS tracking number)

Check Number Enclosed: \_\_\_\_\_ Order Date: \_\_\_\_\_

Comments: \_\_\_\_\_